2025

JERINGAS VACUNADORAS

VACCINATION SYRINGES

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**HISTORY OF STARVET VACCINATION SYRINGES
MAIN CHARACTERISTICS AND DOSAGE
MODELS OF STARVET VACCINATION SYRINGES
ACCESSORIES FOR STARVET VACCINATION SYRINGES
SPARE PARTS FOR STARVET VACCINATION SYRINGES
GUIDE FOR CARRYING OUT GOOD MAINTENANCE AND STERILIZATION**

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Advertisement in the magazine of the Autonomous University of Barcelona (1978)

HISTORY OF OUR STARVET VACCINATION SYRINGES

Our STARVET vaccination syringes were born in Barcelona in the 70s under the name GROTEERMAN and produced by the now defunct company COOPE/MUR, a company created by several partners.

At the end of the 1980s, the COOPE /MUR company was dissolved and GROTEERMAN vaccination syringes were manufactured and marketed by a single owner.

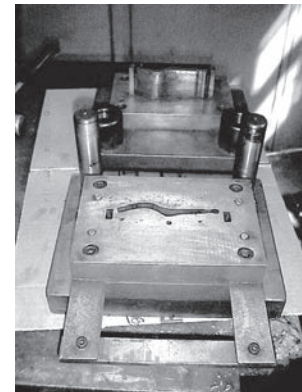
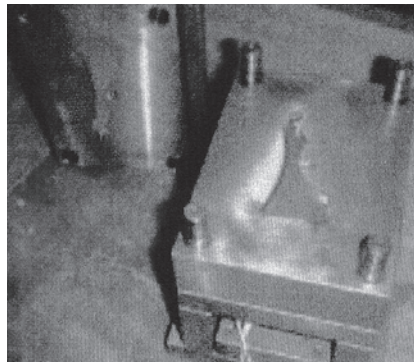
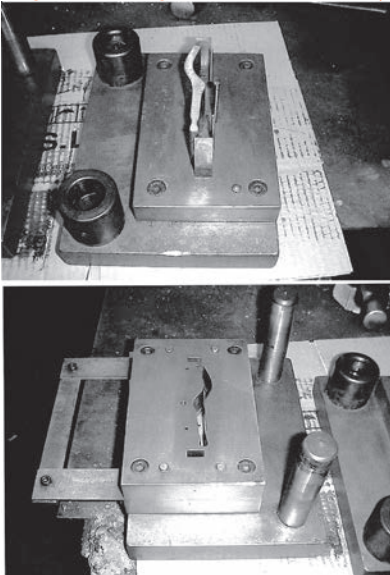
It was not until the mid-90s that this sole owner sold to INSERBO s.l. the rights and machinery necessary for its production. From that moment until today, GROTEERMAN vaccinating syringes are the property of INSERBO s.l. and they became known as STARVET vaccination syringes, preserving the GROTEERMAN name until we decided to use it again in another production line.

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Photographs of some of the molds and dies of our STARVET vaccination syringes

HISTORY OF OUR STARVET VACCINATION SYRINGES

Mr. Sergi Boneu Torres, managing director, founder and owner of INERBO s.l., decided to acquire ownership of these vaccinating syringes because he was aware of their high quality, precision in dosage and ease of handling, since he already offered them to his clients in his long professional career working as a sales representative before forming the company.

Once the vaccinators become the property of INERBO s.l. Our first step was to expand its range with the 1 ml STARVET vaccinating syringe, since until then there were only 5 ml and 2 ml STARVET vaccinators. Since then and until today we work to expand its wide range of accessories by manufacturing all types of extensions, tanks of various capacities, Fishguard protectors for vaccinating fish, cannulas for the insemination of rabbits and much more.

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Turning to the right we increase the dosage
Turning to the left we lower the dosage

MAIN CHARACTERISTICS

STARVET vaccination syringes are characterized by being light, robust, self-refillable, precise in dosage, ergonomic, easy to use, wash and/or sterilize.

They are designed so that the springs of their valve system do not fly off and get lost if we carefully unscrew both the aspiration fitting and the Luer Lock, as it occurs with other vaccinating syringes with the same characteristics that we can find on the market.

Both its glass cylinder and the piston are calibrated and ground, which means that they do not use a rubber plunger and that they self-lubricate with the medication each time they are used.

HIGH PRECISION DOSING

- 1 ml STARVET vaccination syringe has a minimum dosage of 0.05 ml, and its dosage can be increased until reaching the maximum capacity of 1 ml
- 2 ml STARVET vaccination syringe has a minimum dosage of 0.1 ml, and its dosage can be increased until reaching the maximum capacity of 2 ml
- 5 ml STARVET vaccination syringe has a minimum dosage of 0.5 ml, and its dosage can be increased until reaching the maximum capacity of 5 ml

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JG-901 1ml STARVET vaccination syringe with feeding tube and needle

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JG-902 2 ml STARVET vaccination syringe with feeding tube and needle

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JG-905 5 ml STARVET vaccination syringe with feeding tube and needle

MODELOS DE JERINGAS VACUNADORAS STARVET STARVET VACCINATION SYRINGES MODELS

JG-901A Jeringa vacunadora STARVET de 1 ml con portafrascos

JG-901A 1 ml STARVET vaccination syringe with vial holder

JG-902A Jeringa vacunadora STARVET de 2 ml con portafrascos

JG-902A 2 ml STARVET vaccination syringe with vial holder

JG-905A Jeringa vacunadora STARVET de 5 ml con portafrascos

JG-905A 5 ml STARVET vaccination syringe with vial holder

MODELOS DE JERINGAS VACUNADORAS STARVET STARVET VACCINATION SYRINGES MODELS

JG-901B Jeringa vacunadora STARVET de 1 ml con botella

JG-901B 1ml STARVET vaccination syringe with bottle

JG-902B Jeringa vacunadora STARVET de 2 ml con botella

JG-902B 2 ml STARVET vaccination syringe with bottle

JG-905B Jeringa vacunadora STARVET de 5 ml con botella

JG-905B 5 ml STARVET vaccination syringe with bottle

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- JG-901D 1 ml STARVET vaccinations syringe with electronic digital counter
- JG-902C 2 ml STARVET vaccinations syringe with electronic digital counter
- JG-905C 5 ml STARVET vaccinations syringe with electronic digital counter

- JG-901C Jeringa automática con cánula recta para I.A. conejos STARVET, 1 ml
- JG-901C STARVET automatic syringe with straight cannula for rabbit A.I., 1 ml

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All STARVET vaccination syringes are delivered inside a robust and elegant cardboard box with simple but safe opening and closing

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The outside of the box clearly shows its origin (Made In Europe), configuration and capacity

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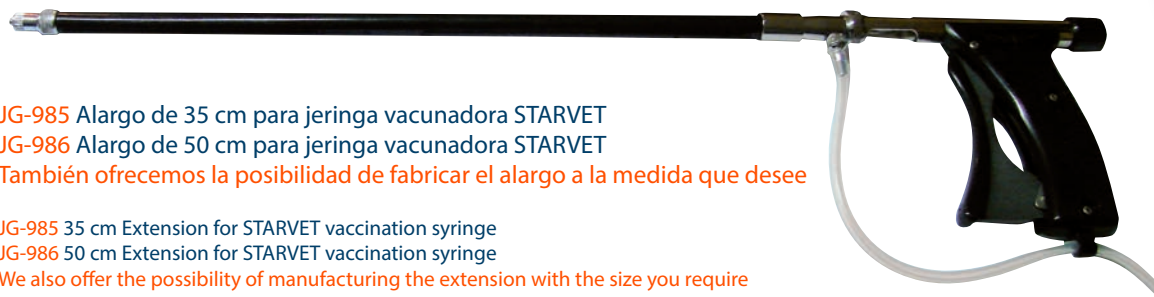
It is supplied with a booklet of instructions for use in which maintenance is detailed, how to proceed for sterilization, a list of optional accessories and all the spare parts that may be required in the future

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They are also delivered along with a bag with spare parts for gaskets and valves and a 13ml bottle of silicone oil for lubrication and care


ACCESORIOS ACCESSORIES

JG-985 Alargo de 35 cm para jeringa vacunadora STARVET
JG-986 Alargo de 50 cm para jeringa vacunadora STARVET
 También ofrecemos la posibilidad de fabricar el alargo a la medida que desee
JG-985 35 cm Extension for STARVET vaccination syringe
JG-986 50 cm Extension for STARVET vaccination syringe
 We also offer the possibility of manufacturing the extension with the size you require

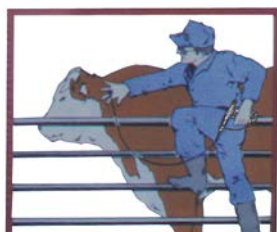JG-987 Alargo STARVET de 1 mt
JG-987 STARVET extension 1 m

JG-390 Alargo vacunador para acoplar a jeringas, STARVET-NEEDLE
JG-390 STARVET-NEEDLE, vaccinator extension to adapt to syringe

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JG-990 Complete container, 3 L for STARVET vaccinators

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JG-990B Set of straps to hold STARVET 3 L containers

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JG-997 Complete STARVET bottle with belt 1 L

JG-996A Cinturón portabotellas STARVET, 1 L (sin botella)

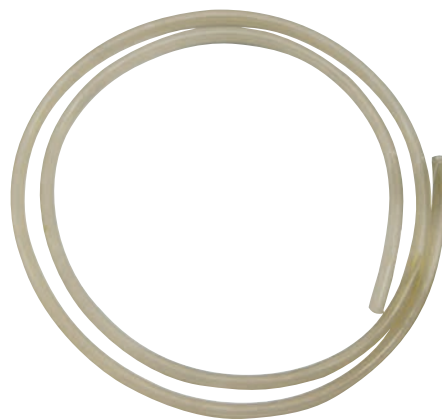
JG-996A STARVET belt bottle carrier 1 L (without bottle)

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JG-996 Complete 1 L bottle with cap for STARVET syringes (without belt)

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JG-995B 100 ml bottle with cap for STARVET syringes

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JG-935A STARVET adapter fitting for STERIMATIC
JG-935B STERIMATIC plastic tube with spring for STARVET vaccinators

JG-999G Protector Fishguard grande de 50 mm STARVET
JG-999F Protector Fishguard mediano de 25 mm STARVET
JG-999E Protector Fishguard pequeño de 15 mm STARVET

JG-999G Large 50 mm STARVET Fisguard
JG-999F Medium 25 mm STARVET Fisguard
JG-999E Small 15 mm STARVET Fisguard

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HYPODERMIC NEEDLES – STAINLESS STEEL

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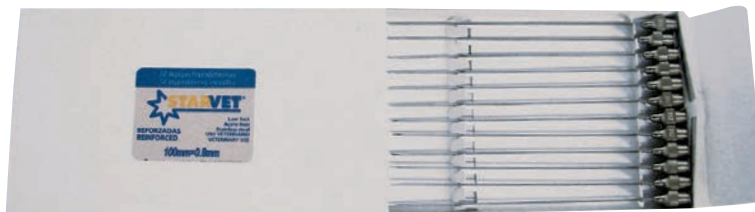
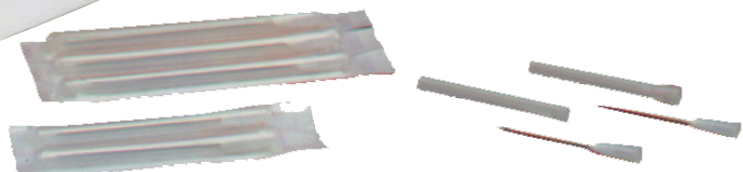
Immediately after use, absolute cleaning is essential for any type of self-refilling syringe that we can find on the market, for our STARVET as well.

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For quick and effective cleaning, we will turn the dispenser knob clockwise until the maximum dosage is reached (fig. 1)

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Place the head of the syringe into a container containing water and dishwashing detergent, in the same proportion as you would wash your dishes.

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With the entire head submerged in the soap and water solution as shown in the image (fig. 2), pull the trigger several times making sure you have completed the entire stroke with each action.

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Once this process is completed and before finishing the cleaning, we will rinse the STARVET vaccinating syringe well, performing the same action, but this time with running water and preferably distilled.

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Place the regulator in the maximum dosage position and separate the head from the rest of the syringe by turning the head nut counterclockwise, separate the head gasket. (fig. 1)

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Gently squeeze the trigger and the piston assembly, along with the glass, will move forward. Separate it from the syringe chassis, ensuring that it does not receive any blows.

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Separate the glass from the piston assembly (fig. 2)

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Place the head, glass and piston assembly in a container of distilled water (to avoid calcium deposits), and boil it for 10 minutes.

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To assemble the syringe after sterilization or after changing over pieces do the same proceeding inversely.

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Display with battery level indicator, total and partial counter of the number of injections

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- JG-910 1 ml with feeding tube and needle
- JG-910A 1 ml with vial holder
- JG-911 2 ml with feeding tube and needle
- JG-911A 2 ml with vial holder
- JG-912 5 ml with feeding tube and needle
- JG-912A 5 ml with vial holder

Compatible con todos los recambios y accesorios disponibles para la vacunadora STARVET manual
Compatible with all the spare parts and accessories available for the manual STARVET vaccinator

JG-913 Batería de recambio

JG-913 Replacement battery

GROTEERMAN®

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INJECTS
WITHOUT CONTACT
WITHOUT NEEDLE

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As we mentioned at the beginning of this catalog, we reserved the name GROTEERMAN for another line of products and here we present it to you.

If you need to perform 0.1ml injections without a needle, our GROTEERMAN intradermal inoculation syringes are ideal for IDTB (Intradermotuberculinization) in Bovine and tested by the Veterinary Health Surveillance Center (VISAVET) and for vaccination against Myxomatosis in rabbits

JG-900 Jeringa intradérmica GROTEERMAN

JG-895 Jeringa intradérmica GROTEERMAN con cabezal de 3 salidas

JG-900 GROTEERMAN Intradermal syringe

JG-895 GROTEERMAN Intradermal syringe with 3-outlet head

JG-890 Cabezal de 3 salidas para jeringas GROTEERMAN

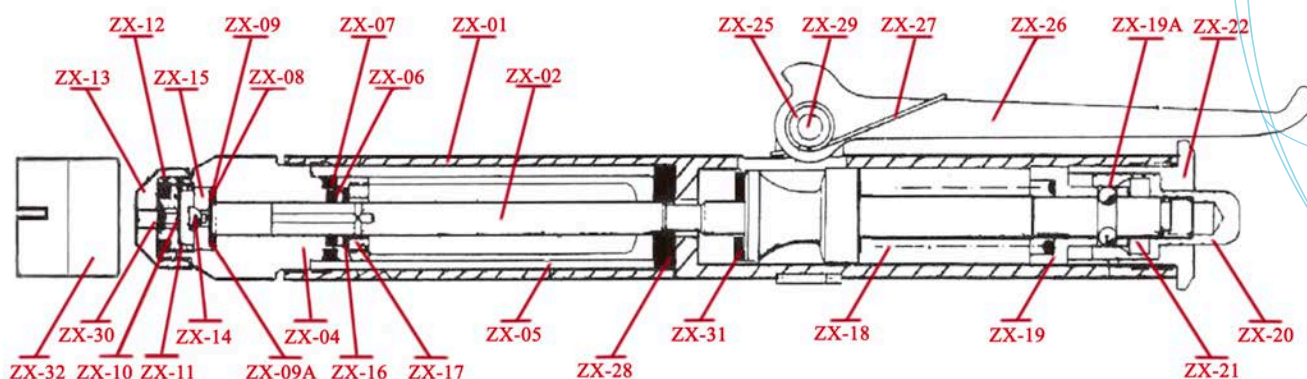
JG-890 3-exhaust head for GROTEERMAN syringues

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JG-900A GROTEERMAN Intradermal automatic syringe

JG-900B GROTEERMAN Intradermal automatic syringe with 3-outlet head

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